CT-BED-MAT

Under Mativess Ded Pressure Mat
(((CareTech.
Instructions
LUIt the mattress LUIt the mattress Place the Bed Pressure Mat Unis side up, under the mattress as indicated below. Avoid any bumps, rivets or hinges
that may affect the Mats sensitivity Krep the cable clear of any hinges and the bed rail mechanism sp uses a s not damaged during the operation of the bed.
Place the mattress down on the bed base. Make sure the bod linen does not bunch or gather on the top of the Mat Plag the extension cable into the Mat and the main form
In the case that the mattreas is constructed differently, use layers of town in a viryl case with a typer. The Mat can then be positioned between the layers of foarm to increase its smooth-Ny
Test the bed service for operation, by pressing on the matter as in the movimend to adjust the peolision of the matteres or Matter for optimal position
and the state of t
Care Instructions
Donot bend the Bed Pressure Mat Store vertical or flat
Do not immerse the Mat into water or any Rold Crean the Mat with a warm 0 pdochr istan

Instructions:

- 1) Lift mattress
- 2) Place mat under mattress. Avoid any bumps, rivets or hinges that may affect the mats sensitivity.
- 3) Keep the cable clear of any hinges and the bed rail mechanism.
- 4) Place the mattress down on the bed base. Make sure the bed linen does not bunch or gather on top of the mat.
- 5) Plug the extension cable into the mat and the wall socket.
- 6) In the case that the mattress is constructed differently to a conventional one use layers of foam in a vinyl case so that the mat can be positioned between the layers of foam to increase the sensitivity.
- Test the bed sensor for operation by pressing down on the mattress. Adjust the position on the mat accordingly for the optimal position.

Care Instructions:

- 1) Do not bend the mat.
- 2) Store vertically or flat.
- 3) Do not immerse the mat into water or liquid.
- 4) Clean the mat with a warm damp cloth and standard cleaning products.